


## How to connect your IOS to Go Digital Dental

### 3Shape Communicate

- 1 Log in to <https://portal.3shapecommunicate.com/login>
- 2 Click connections at the top of the page
- 3 Add connection
- 4 Enter the lab name **Go Digital Dental** or email [scans@godigital-dental.com](mailto:scans@godigital-dental.com) and search
- 5 Once you can see the correct laboratory click Connect

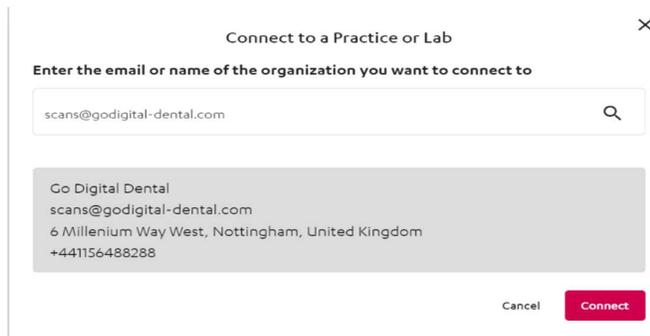

- 6 Once your request has been sent to the lab, we will accept within 24hours.

### CS Connect

- 1 Login to CS Connect <https://csdentalconnect.com/>
- 2 In the toolbar, click 
- 3 In the Partners window, click Invite a partner
- 4 Enter Go Digital email [technician@godigital-dental.com](mailto:technician@godigital-dental.com)
- 5 In the Message box, either leave the default text or write your own message
- 6 Click Send
- 7 Once your request has been sent to the lab, we will accept within 24hours

### Sirona Connect via the Cerec Portal

- 1 When you are in the Portal, **click on the (+) in the Recipients tab**. It will take you to the [customer.connectcasecenter.com](https://customer.connectcasecenter.com)
- 2 Under **My Favourite Laboratory** click on **Search Recipient**
- 3 Enter **Go Digital Dental** or **NG86AS** and search
- 4 Scroll down to see the Search Results. Click on the **orange (+)** next to **Go Digital Dental** to add to your Account
- 5 Once you have selected Go Digital Dental you have added the lab to the portal successfully

### Sirona Connect via Cerec Connect




- 1 Go to <https://customer.connectcasecenter.com/> and log in
- 2 Under **My Favourite laboratory** click on **Search Recipient**
- 3 Enter **Go Digital Dental** or **NG86AS** and search
- 4 Scroll down to see the Search Results. Click on the **orange (+)** next to the **Go Digital Dental** to add to your Account
- 5 Once you have selected Go Digital Dental you have added the lab to the portal successfully

## iTero

- 1 Contact iTero Customer support on **0800 101 7005**
- 2 Provide Go Digital ID number; **1448622** to establish certified connectivity between your practice and Go Digital Dental.

## Medit

- 1 Log in to [www.meditlink.com](http://www.meditlink.com) with the administrator account
- 2 Go to Partners on the left menu
- 3 Search for the lab that you wish to make a partnership with
  - a. You can search for partners by their lab name, email address, postal code, or address. Make sure to use a valid email address format: `example@mail.com`
  - b. If you enable the Show Map feature, it will show search results within 5km ~ 1,000km via Google Maps

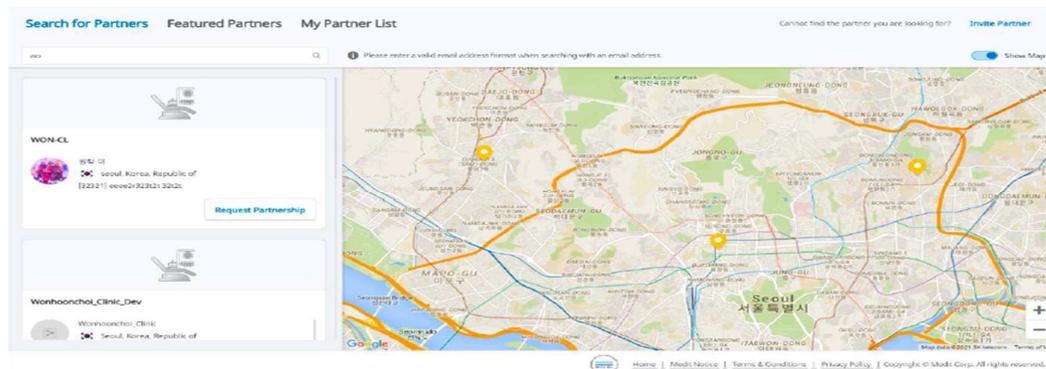

- 4 Locate the lab and click the **Request Partnership** button. Check the partner's terms & conditions, and click the **Confirm** button
- 5 You will see that the lab has been added to the Partners > My Partner List with the status reflected as **Partnership Pending**.
  - The "Partnership Pending" status will be changed to "Partnership" when the lab accepts the request.

Alternatively, you can upload files via our online dentist portal <https://www.godigital-dental.com/dentist-portal/dental-lab-file-upload/>

For any other technical support please email [technician@godigital-dental.com](mailto:technician@godigital-dental.com) or WhatsApp 01156488288

